

THE Face Shields

THAT SAVED THE DAY!



Brought to you by your rp+m superheroes:

KEEP GERMS AT BAY BY CLEANING THE PROPER WAY!

When disinfecting or sanitizing your rp+m Face Shields, proper procedures should be taken. To minimize the risk of damage, the following cleaning agents are recommended for this product:

- sBioMed, LLC Steriplex SD
- Safetec of America Safetec surface wipes
- CONTEC Healthcare Perodox RTU
- CONTEC Healthcare Prostat sterile wipes PS-911EB
- Generic hydrogen peroxide - pharmacy grade
- Generic isopropyl alcohol - diluted with water to 30% strength
- Generic hydrogen peroxide 28%, isopropyl alcohol, soap solution

SPECIAL NOTES!



- Do not use unapproved cleaners or disinfectants on sheet products.
- Generally speaking, soap and water, or hydrogen peroxide or isopropyl alcohol-based sanitizers work best as disinfectants on these products.
- Disinfectants containing hydroxides, dimethyl benzyl or ethyl ammonium chloride, ethylenediamines and hypochlorite will crack the polycarbonate sheets.



THANK YOU
for supporting our small-but-mighty Ohio-based company!

To order more rp+m Face Shields,
please contact us at orders@rpplusm.com
or call 440.930.2015